



## **Instruction Sheet Tube Swing Kit 1501**

### Instruction Sheet Tube Swing Kit 1501

Store these instructions in a safe place or with the enclosed maintenance checklist. Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using OT4YOU Equipment. **Warning:** Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact OT4YOU.

**IMPORTANT:** Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact OT4YOU immediately.

#### **MATERIALS LIST:**

Tube Swing Kit 1 - Tube Swing Harness 1 - Valve Stem Cover 1 - Truck Tube 1 - Instruction Sheet 1 - Maintenance Checklist

#### **MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED:**

Minimum Height required is 8 ft. Keep a clear distance of 6 ft. from walls and other obstacles.

**HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED:** Use of the Height Adjuster is recommended for ease of adjusting equipment. A Safety Rotational Device must be used to prevent damage to your harness.

**WORKING LOAD:** 60kg. when used with our tube.

**ASSEMBLY PROCEDURES:** • Inflate tube to 42" on outside diameter with an inside diameter of 21". • Wrap harness around the tube with the foam padding on the inside of the cover covering the valve stem. The Blue Webbing should go around the tube completely • To hang your tube swing put both long blue loops through a Safety Snap and hang the safety snap from your suspension system. You must use a Safety Rotational Device with your tube swing. This will give you free movement and take stress off of your support system.

**MATTING PRECAUTIONS:** • Always use appropriate matting.



## MAINTENANCE CHECKLIST

All of the following should be checked on a monthly basis and recorded on the checklist enclosed. • Check for excessive wear on the blue loops where they contact the safety snap. If excessive wear or fraying is found discontinue use until repaired. • Check that valve stem is not poking through cover.

**STORAGE PROCEDURES** • Store out of direct sunlight. • Keep away from chemicals and household cleaners. • Store where not accessible by clients without proper supervision.

**CLEANING INSTRUCTIONS** • Check with your infectious disease department before using these procedures. • Check for client allergies first. • Clean tube and harness with a damp cloth using soap and water or disinfectant. • Dry before using.

**CAUTION** ♦ Check ceiling eye bolt for wear and movement. Use a Safety Rotational Device to prevent abnormal wear. ♦ Floor mats of proper thickness should be positioned properly under and around all suspended equipment. **Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. OT4YOU equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision. This sheet is for example only. It is not meant to, in any way, replace local, regional or national standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

